
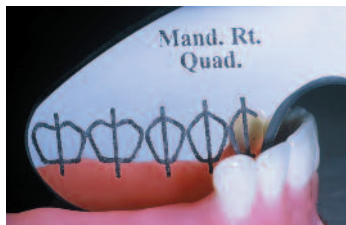


INSTRUCTIONS FOR USE

1. Choose the proper quadrant on the Guide. All four quadrants are on each Guide.
2. Superimpose the laser-etched teeth of the Guide on the edentulous ridge using any remaining natural dentition as an additional orientation for the Guide.
3. Align implant to be placed with the horizontal lines of the Guide to ensure parallelism and alignment.
4. Use Guide after each implant is placed to verify alignment and parallelism.

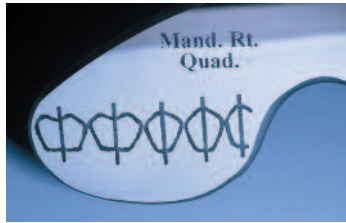

GUIDE POSITIONING BY QUADRANT

- MAXILLARY RIGHT SIDE:** Mirror is placed palatal to the ridge and the operator looks from the facial side.
- MAXILLARY LEFT SIDE:** Mirror retracts the cheek and the operator views from the palatal side.
- MANDIBULAR RIGHT SIDE:** Operator views from the facial side and the mirror retracts the tongue.
- MANDIBULAR LEFT SIDE:** Mirror retracts the cheek and operator views from the lingual side.

CLEANING/STERILIZING INSTRUCTIONS

Garfinkel Guides are coated with rhodium, a durable metallic finish. Care should be taken to ensure both proper cleaning and longevity.

The recommended method for cleaning is steam autoclaving using the following procedure:

1. Hand wash the mirrors with clean gauze and germicidal soap containing 2% chlorhexide.
2. Dry the Guide using gauze sponges or a Zeza Multi-Purpose Surgical Towel. Wrap the Guides in a surgical towel taking care to separate them with layers of towel.
3. Place in a steam autoclave unit and process.

Chemical sterilants — We do not recommend the use of chemical or cold sterilants. These chemicals may leave permanent streaks on the surface of the Guide.

